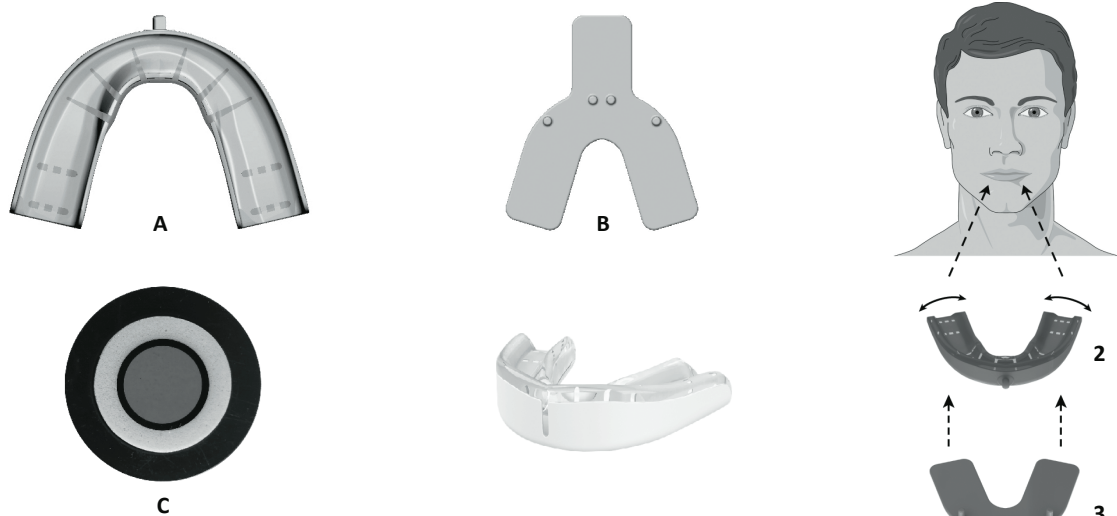
Oscimed SA
Fritz-Courvoisier 105
CH-2300 La Chaux-de-Fonds
info@oscimedsa.com

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Instructions for use
Dental protection against erosion due to bruxism

Medical device Thermoforming and operating instructions

Package contents

- One flexible thermoformable bite splint for the upper jaw (A)
- One toothprint plate (B)
- A water temperature indicator (black disc with green centre) (C)
- One ventilated storage box
- A set of instructions

Indications
Protects teeth against erosion due to bruxism, grinding and clenching of the teeth.

Contraindications
Severe periodontal disorders.
Fewer than 8 teeth on the upper arch. If the teeth are unstable and fragile or brittle teeth, ask your dentist for advice. Users under 18 years of age.

Possible side effects

- Tooth sensitivity.
- Hyper-salivation, decreasing after a few days of regular use, dry mouth, possible gag reflex very rarely observed (which disappears quickly).

⚠ If uncomfortable side effects persist, discontinue wearing the device and seek medical advice.

In the event of a serious accident, please contact the manufacturer or the competent authority.

Maintenance, Product Lifespan, Warranty
In the morning, remove the splint, rinse it and keep it throughout the day in a container of lukewarm water with an antibacterial, anti-tartar tablet for dentures, rinse thoroughly with clean water before putting the splint back into your mouth for the night. The lifespan of the device depends on its maintenance, the acidity in the mouth and the severity of the teeth clenching (bruxism). The device is guaranteed only in case of manufacturer defects.

⚠ Inspect daily the condition of the orthosis. The device MUST be changed if cracks, breaks, or substantial deterioration of the material occur.

Adaptation to the teeth (thermo-forming)

1• Place the temperature indicator in a saucpan containing 7 to 8 centimetres of cold water and heat the water. Turn off the heat source when the centre of the temperature indicator changes from green to red. Let the split sit in the hot water for 3 minutes.

IMPORTANT: Do not leave the splint in boiling water.

IMPORTANT: The interval between removal from the saucpan and taking the impression must be limited to 10-15 seconds.

2• Sit or stand facing a mirror. Take the splint out of the hot water using a spoon and shake it to remove any water. Place the splint aligning the central line of the splint with the central gap between the superior central incisors. Adjust the width of the splint (Fig. 2) to the jaw size so that the molars are inserted correctly.

3• Hold on the splint with one hand and insert the toothprint plate in mouth with the other (Fig. 3). Bite hard to make the dental impression. Keep your teeth close together and press your fingers against the sides of the splint for a perfect impression of the sides of your teeth. Release the pressure on the teeth and rinse your mouth with cold water. Let cool 2 minutes in your mouth.

4• Before going to bed, or if you are experiencing stress, put in your splint.

Impression quality
The comfort and grip of the splint depend on the shape of the teeth, the number of teeth and the quality of the impression taken. Check that you have taken a good impression of the shape of your teeth on the splint and if not, re-take the impression, correctly following the temperature instructions and using enough force to shape the device. Any parts of the thermoformable material causing discomfort can be cut off.

⚠ Any other modification is strictly forbidden.

⚡ Store away from light. At the end of its life, throw away the device.

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